





Ankle Pro Smart Wrap™ Quick Start Guide

Setting Up the Kelvi Pro^e Console

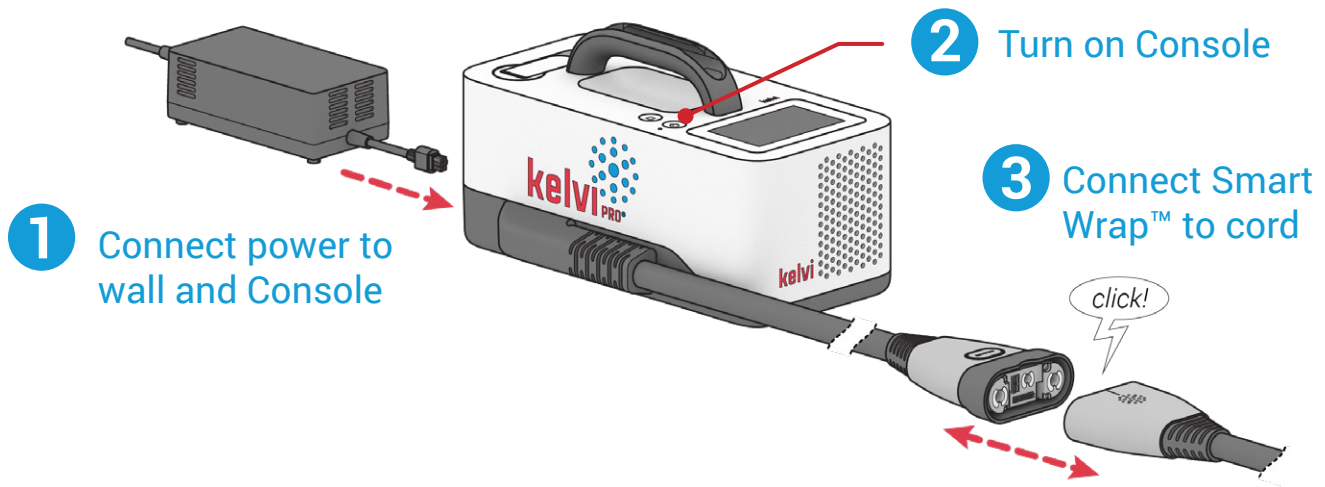


Applying the Smart Wrap™


- 4** Hold the Smart Wrap™ by the loop and slide the foot in. Position the heel at the rear of the pocket. Do not slide toes forward.

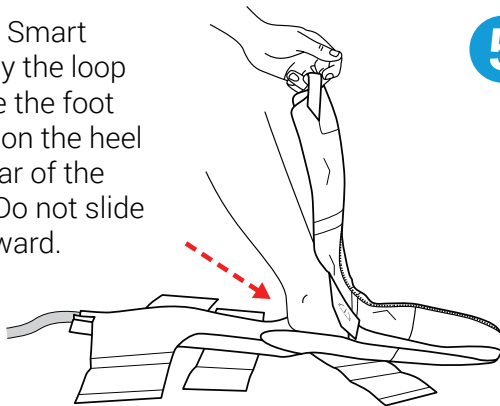


- 5** Support the back of the Smart Wrap™ and secure the upper straps horizontally across the front of the shin.

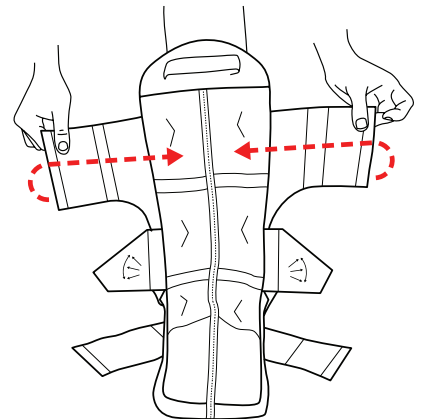


- 6** Attach the wings. Adjust the angle of the wings for best comfort and fit.

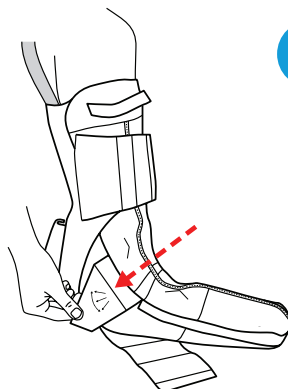

- 7** Secure the middle and lower straps. Adjust all straps as needed for comfort, keeping the heel firmly in place against the back of the Smart Wrap™

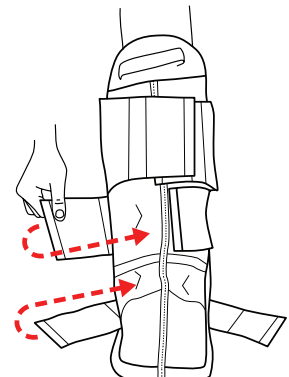


Setting Up Treatment*

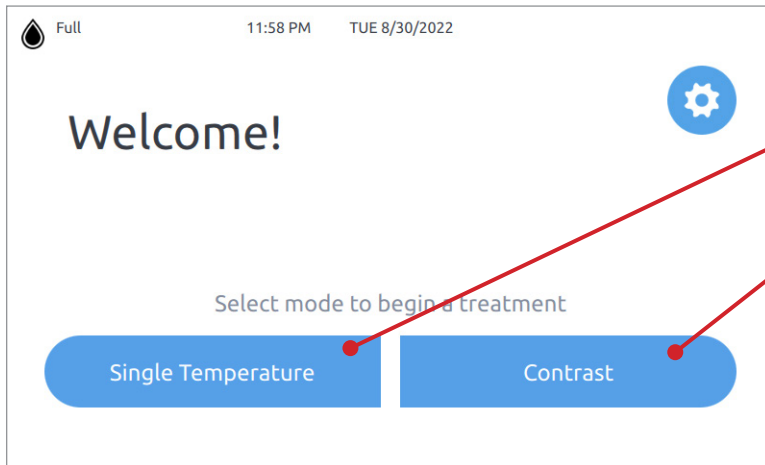

8 Select treatment type

On the touchscreen, select **Single Temperature** for constant treatment

Select **Contrast** for alternating cool and hot treatment

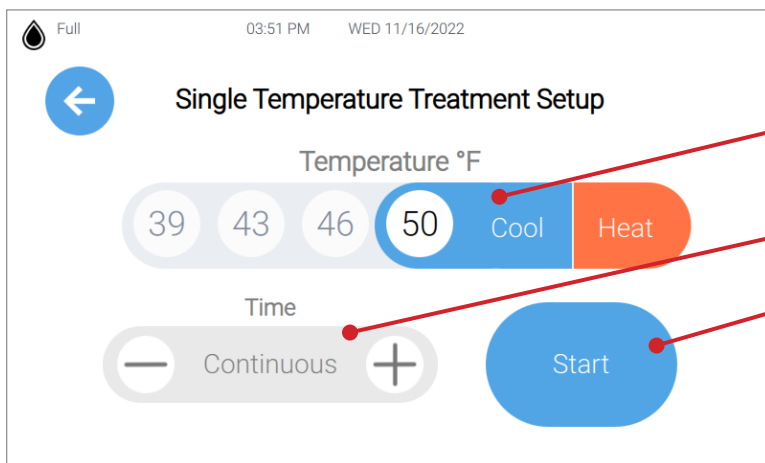


9 Set temperature and time

Select **Cool** or **Heat**, then select temperature.

Set a time limit or choose **Continuous**.

Press **Start** to begin treatment.

Smart Wrap™ may be adjusted as needed for comfort during treatment.

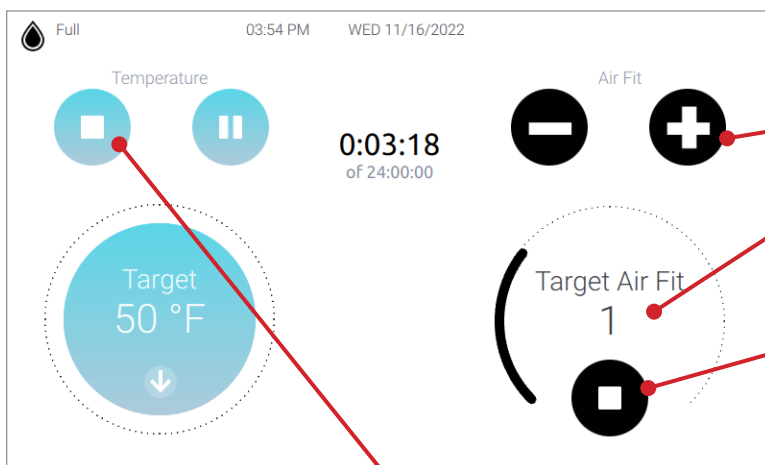

10 Adjust air pressure for fit

Select **+** or **-** to change air pressure.

Start low and increase by one level at a time. Do not exceed comfortable pressure.

Press **Stop** to release air pressure.

Press **Stop** to end treatment.

* Follow clinician prescription for treatment



Scan here for complete instructions for use



IMPORTANT: Read complete instructions including indications, contraindications, cautions, and warnings before using this product. For assistance in using the Kelvi Pro™ System, contact your licensed or certified healthcare professional.

