## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### INTENDED USE/INDICATIONS:

The AirCast A60™ Ankle Support is designed to provide support and protection for the ankles. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction. Providing mild protection of a limb or body segment for acute and prophylactic care.

### CONTRAINDICATIONS: N/A

### WARNINGS AND PRECAUTIONS:

- Do not use over open wounds.
  - If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
  - Do not use this device if it was damaged and/or package has been opened.
- Note:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### APPLICATION INFORMATION:

- Put on an absorbent sock**
  - Unfasten strap from black plastic ring, open top flaps, and open brace
  - Slide foot, toe first, into top of brace (Fig. 1)
  - Pull brace stabilizers upward and align stabilizers with sides of ankle
- Wrap flaps around top of ankle and secure (Fig. 2)**
- Insert end of strap into black plastic ring**
  - Bend knee and align over toes
  - Pull strap until snug and secure strap end onto body of strap (Fig. 3)

### CLEANING INSTRUCTIONS:

The entire A60 Ankle Support can be hand washed in lukewarm water with mild soap and air dried.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

### FOR SINGLE PATIENT USE ONLY

**NOT MADE WITH NATURAL RUBBER LATEX.**

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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