ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user profile is the patient, the patient's carer or a family member providing medical assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS:

The DonJoy Genuforce is designed to support the knee. It may be suitable for use following mild soft tissue injuries and may help those suffering from chronic instability. Soft-good/semi-rigid designed to restrict motion through elastic or semi rigid construction.

FEATURES:

- A silicone buttress around the knee reduces the pressure level and Relieves the patella.
- Helps give the patella slight guidance.
- Wrinkle-free, non-slip fit for comfort of wear.
- 3-D anatomically contoured knit provides compression.
- Breathable material

INDICATIONS:

- Post-operative or post-traumatic use, chronic soft tissue inflammation of the knee joint
- Meniscus tears
- Feelings of instability
- Meniscus injuries
- Ligament injuries
- Load-related pain in the knee
- Swelling

CONTRAINDICATIONS:

Do not use if you are allergic to any of the materials contained in this product.

WARNINGS AND PRECAUTIONS:

- If pain, swelling, changes in sensation or other unusual reactions occur during this product, you should contact your doctor immediately.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or package has been opened.
- The GENUFORCE® Knee Support should only be used after consulting a health care professional. Do not use the following device for skin diseases and injuries in the knee area (e.g. allergy related skin changes, irritations, scars with swelling, redness, hyperthermia). Varicose veins with reflux disorders. Lymphatic disorders, indistinct soft tissue swelling around the knee area or circulatory or sensory disorders in the legs.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device

APPLICATION INSTRUCTIONS:

1. Sit down to fit GENUFORCE®.
2. Grip the knee support with both hands by the upper blue strap.

3. Pull the support over the knee, so that the circular silicone buttress covers the kneecap evenly.
4. The silicone buttress should be positioned so that its elongated part sits below the mid-kneecap.

USAGE INSTRUCTIONS:

- The support is worn directly on the skin. Therefore, the skin must have no injuries and should be free of moisture, oils, creams, gels or similar residues when putting on the device.
- The support is intended for single patient use only.
- The support should not be worn for more than 8 hours a day.
- With proper use and the observance of all instructions, no pressure points should occur.
- The manufacturer shall bear no responsibility for any changes to the product or incorrect usage.

CARE:

- Hand wash in lukewarm water (max. 30 °C) with a mild detergent.
- Air dry (do not place on radiators/heaters or other heat sources).
- Do not use bleach or chlorine, do not iron.

COMPOSITION:

Knitted material: polyamide, elastin, polyester / Buttress: silicone / Cover: terrycloth

LATEX CAUTION: THIS PRODUCT CONTAINS NATURAL RUBBER LATEX WHICH MAY CAUSE ALLERGIC REACTIONS.

FOR SINGLE PATIENT USE ONLY.

GARANTY DJO, LLC WILL REPAIR OR REPLACE ALL OR PART OF THE UNIT AND ITS ACCESSORIES FOR MATERIAL OR WORKMANSHIP DEFECTS FOR A PERIOD OF SIX MONTHS FROM THE DATE OF SALE.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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WARNINGS AND PRECAUTIONS :

- Si pain, swelling, changes in sensation or other unusual reactions occur during this product, you should contact your doctor immediately.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or package has been opened.
- The GENUFORCE® Knee Support should only be used after consulting a health care professional. Do not use the following device for skin diseases and injuries in the knee area (e.g. allergy related skin changes, irritations, scars with swelling, redness, hyperthermia). Varicose veins with reflux disorders. Lymphatic disorders, indistinct soft tissue swelling around the knee area or circulatory or sensory disorders in the legs.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device

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