ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:
THE USER SHOULD BE ABLE TO:

- READ AND UNDERSTAND THE DIRECTIONS, WARNINGS AND CAUTIONS.

INTENDED USE/INDICATIONS: Subtle Cavus Foot (higher arches and heel varus).

CONTRAINDICATIONS: None

WARNINGS AND CAUTIONS: If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

APPLICATION INFORMATION:

Arch Rival Sizing:

Approximate sizing is determined by shoe size. Due to the unique design of the Arch Rival, please refer to the fitting guide for precise instructions. Arch Rival fits in most athletic and walking shoes.

1. Appropriate fit allows your 1st metatarsal head to sit in the recessed area with your heel comfortably placed in the back of the orthotic. **Note:** You may need to size up or down for optimal fit. (Figure 1)
2. Using the insole from your existing shoe as a template, trim the Arch Rival with scissors to match the size. (Figure 2)
3. Place securely into shoe, ensuring the Arch Rival is seated properly and toe area is not folded over.

Break-In the Arch Rival Orthotics:

Allow 1-3 weeks to properly break-in the orthotics.

For first time use, do not exceed 1-2 hours. If there are no adverse effects, increase use by 1-2 hours per day until full time use.

Sizing Chart (Shoe Size)

	A	B	C	D	E	F	G
WOMEN	4.5 - 6	6.5 - 8	8.5 - 10	10.5 - 12	12.5 - 14	14.5 +	-
MEN	-	5.5 - 7	7.5 - 9	9.5 - 11	11.5 - 13	13.5 - 15	15.5 - 17

CLEANING INSTRUCTIONS: Use a soft brush with anti-bacterial soap and warm water.

Air dry completely before reinserting into shoe.

Store in a cool, dry place.

Do not machine wash or dry.

 Do not wash.

MATERIAL CONTENTS:

Top Cover Fabric- Nylon with XS Silver Infusion.

Top Cover Cushion & Shell - EVA.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of three months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

INTENDED FOR SINGLE PATIENT USE.
NOT MADE WITH NATURAL RUBBER LATEX.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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