

**PROCARE**

### Adult Sizing Guide Transitional 172 Cervical Collar

Please refer to the ProCare® Instructions for Use for more information.

**READ INSTRUCTIONS BEFORE USE:**  
Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.  
**FOR PRODUCT SUPPORT PLEASE CALL 888.405.3251 OR EMAIL [PRODUCT.SPECIALIST@DJOGLOBAL.COM](mailto:PRODUCT.SPECIALIST@DJOGLOBAL.COM).**

Part No.	Description	Size	Height	Neck Circ.	Color Code
79-83288	Transitional 172 Collar	Adult X-Tall	4.5 in	13-22 in	Yellow
79-83287	Transitional 172 Collar	Adult Tall	3.75 in	13-22 in	Green
79-83285	Transitional 172 Collar	Adult Regular	3.0 in	13-22 in	Orange
79-83283	Transitional 172 Collar	Adult Short	2.25 in	13-22 in	Blue
79-83368	Transitional 172 Collar w/ Replacement Pad Set	Adult X-Tall	4.5 in	13-22 in	Yellow
79-83367	Transitional 172 Collar w/ Replacement Pad Set	Adult Tall	3.75 in	13-22 in	Green
79-83365	Transitional 172 Collar w/ Replacement Pad Set	Adult Regular	3.0 in	13-22 in	Orange
79-83363	Transitional 172 Collar w/ Replacement Pad Set	Adult Short	2.25 in	13-22 in	Blue

### Sizing Guidelines



DJO, LLC | A DJO Global Company  
800.336.6569 | 760.727.1280 | 800.936.6569  
1430 Decision Street | Vista, CA 92081-8553 | U.S.A.  
[DJOGlobal.com](http://DJOGlobal.com)

*Together in Motion.*

©2013 DJO, LLC

13-7269 REV A

**PROCARE**


### Adult Sizing Guide Transitional 172 Cervical Collar

Please refer to the ProCare® Instructions for Use for more information.

#### Select the Right Size

Position the patient into neutral head alignment, place the guide on the highest point of the trapezius and flat against the side of the head.

Visualize an imaginary horizontal line from the bottom of the chin back to the sizing guide. The perpendicular line will indicate the proper collar size.

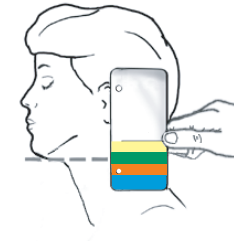

#### Check to Ensure a Proper Fit

1. The chin must be flush with the front of the chin support on the front panel.
2. The flexible tabs on the front and back panels should be bent slightly outward to promote a better fit.
3. The collar should be "snug" but not touch the tracheal area.

**Extra Tall**

**Tall**

**Regular**

**Short**

**IMPORTANT:**

Proper training and education in the use of this device should take place before it is applied. These instructions and guidelines are not offered as medical recommendations.