


UADRANT **ULTRA**SLING™

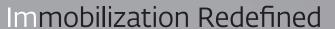



Patient Application Instructions

Apply Brace



A. Slide Arm Brace under injured shoulder, with Hand Support in extension.

Note: See Brace Removal to position armsling as shown with hand support in extension.



Apply Bicep Strap and Forearm Strap. Bend elbow to lock in place. Apply optional Wrist Pad. Apply Hand Strap.



Insert hand into Waist Belt Pocket to apply Waist Belt. Center and secure Belt Closure Tab to Front Panel. Tighten Belt Adjustment Strap if necessary.

Note: The brace should be situated with lower portion of Rigid Frame sitting approximately 1-2" above belly button and Arm Brace aligned with shoulder.

Brace Removal: Undo Hand, Forearm, and Bicep Straps. Push Hand Support in. Extend arm. Release Belt Closure Tab and remove waist belt. Slide brace off. Note: Remove optional wrist pad if necessary.





For more information including application videos, please visit

DJOglobal.com/donjoy/ultrasling-quadrant

USE AND CARE: Separate the rigid components from the soft goods prior to cleaning. Hand wash in water 86°F (30°C) using mild soap. Air dry.

MATERIAL CONTENTS: Nylon Aluminum Stainless Steel Zinc coated steel wire Copper Nylon woven Polyurethane Foam, Silicone, Epoxy, Polyurethane. **NOT MADE WITH NATURAL RUBBER LATEX. CONTRAINDICATIONS:** None

Next Appointment	1 1
Date:	
Time:	

For product assistance, please contact Product Support at 1.888.405.3251 or email product.specialist@djoglobal.com

Please read the Instruction for Use prior to using this device. Adjustments can only be made by a licensed professional.

Individual results may vary. Neither DJO Global, Inc. nor any of its subsidiaries dispense medical advice. The contents of this poster do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.

DJO, LLC | A DJO Global Company

T 800.336.6569 **D** 760.727.1280 **F** 800.936.6569 1430 Decision Street | Vista, CA 92081-8553 | U.S.A.