EXOS OVEN MANUAL

ENGLISH	1
ESPAÑOL	9
DEUTSCH	17
ITALIANO	25
FRANÇAIS	33
NEDERLANDS	41
SLOVENČINA	49
DANSK	57
SVENSKA	65
SUOMI	73
ČEŠTINA	81
PORTUGUÊS	89
NORSK	97
简体中文	105
日本語	113

TABLE OF CONTENTS - *English*

Warnings & Precautions	2
Assembly Diagram	4
Operation Instructions	5
Caution	6
Cleaning and Care Instructions	6
Specifications	6
Warranty Information	7
RoHS Restricted Materials	8
Material & Toxicological Profile Maximum Concentration	8

WARNINGS & PRECAUTIONS

WARNING

WHEN USING ELECTRICAL PRODUCTS, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED.

WARNING! Read and understand all warnings and Instructions for Use before using this device.

DANGER!

To reduce the risk of electrocution:

1. Read all instructions, safeguards, and warnings before using the appliance.
2. Do not place appliance where it can fall or be pulled into water or other liquids.
3. Do not reach for an appliance that has fallen into water. Unplug immediately!
4. Do not immerse cord, plug, or power head in water or other liquids.
5. Do not operate the oven in the presence of flammable substances as this would constitute an explosion risk.
6. The oven must be mounted on non-flammable surface material.

WARNING!

To reduce the risk of burns, electrocution, fire, or injury:



1. Use this appliance for its intended purpose. Do not use any other accessories or attachments not recommended by the manufacturer. They may result in fire, electrical shock, or personal injury.
2. Never operate this appliance if it has a damaged cord or plug, is not working properly, has been dropped or damaged, or dropped into water.
3. Keep the cord away from heated surfaces.
4. Always unplug power cord by pulling on the plug. DO NOT unplug by pulling on the cord.
5. Never force the plug into an outlet.

WARNINGS & PRECAUTIONS - *(continued)*

6. Do not operate outdoors or operate where aerosol (spray) products are being used.
7. Do not let cord hang over the edge of the table or counter.
8. Do not place near hot gas or electric burner or in a heated oven.
9. Do not move the appliance while it is hot.
10. Before removing plug from the wall, turn the unit off first.
11. Check all electrical wiring Beware of damaged cord or plug.
12. Do not disassemble the product There are not user serviceable parts.
13. Do not leave the appliance unattended while in use.
14. Use the handles when lifting the dome from the base.
15. If the power cord is damaged, it should be replaced by a special cord or assembly from the manufacturer.
16. Keep this manual handy for easy future reference.
17. Use with adequate electrical system.
18. All metal items in the oven, such as the liner pan and rack can get very hot during heating. Please be careful to avoid touching these items when the oven is hot.
19. Allow liner pan and rack to cool completely before cleaning.
20. Do not place anything on or near the oven that could block the vents.
21. Do not place the oven in a position whereby the vents on the sides and top of the oven could be blocked. Maintain minimum clearances of 6" on all sides and 12" above the oven.
22. The oven should be positioned so that the power cord can be easily disconnected.

ASSEMBLY DIAGRAM

Power Head with Handles and Power Controls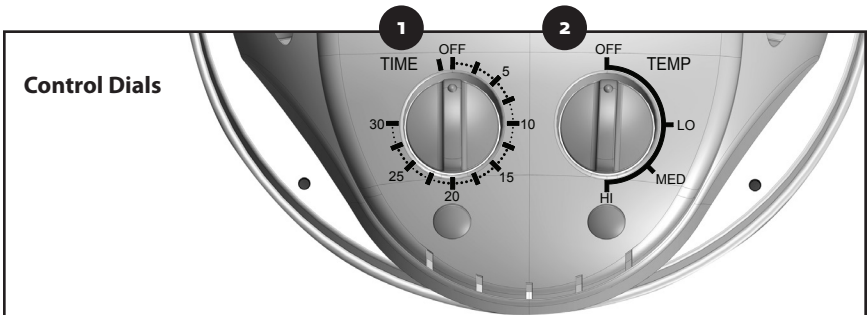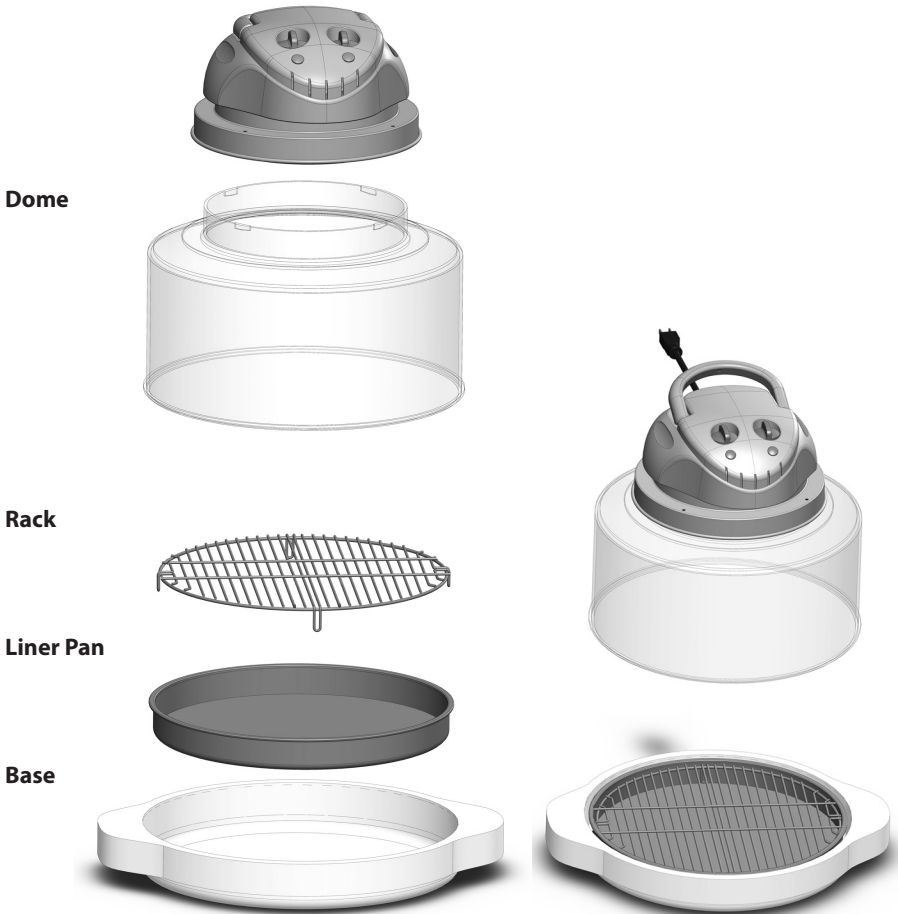


OPERATION INSTRUCTIONS

When used for heating Exos products.

1. Read Exos Product instructions before applying the product.


2. Remove product from package and open brace so that sides do not overlap. *Preheating oven is not required.*

3. Set oven temperature. Set between MED and HI TEMP for best results.
Note: Handle must be in down position for oven to turn on.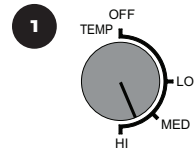

4. Turn timer dial to 3 - 5 minutes. Please see product instructions for recommended heating time.

5. **Note:** It is recommended to first turn past the 10 minute mark, and then back to 3 - 5 minutes to ensure proper operation of the timer.

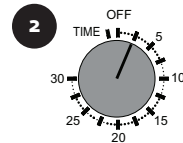5. Remove product from oven when bell rings.

CAUTION

- Do not heat items longer than 15 minutes.
- Do not touch the dome or any surface on the heating area during operation. The air inside the dome can reach 225°F which makes the surfaces very hot and may cause burns.
- Do not open the dome or remove the power head while it is still operating. Turn the Timer dial to "OFF" first to turn off the power before removal.

CLEANING AND CARE INSTRUCTIONS

General Instructions:

- Unplug the unit and wait until cool before cleaning.
- For all parts, other than the power head, disinfect by wiping with alcohol or wash with mild soap and water then dry with towel or allow to air dry.
- Protect the metal and plastic surfaces; do not use abrasive cleaners.
- The outer surface of the power head can be cleaned by wiping it with a damp cloth.
- Do not submerge the power head.
- Avoid using any cleaning agents that could cause a hazard as a result of a reaction with parts of the equipment or with material contained in it.

SPECIFICATIONS

Intended use:

This oven is intended to be used to heat items to approximately 200 degrees Fahrenheit.

Specifications

Dimensions: 15.4" x 14.5" x 15"

Weight: 9.1 lbs.

Power input 120V version: 120 Vac 60 Hz 11 A

Power input 230V version: 230 Vac 50/60 Hz 6 A

Standards: EC 61010-1, 2nd Ed.; CSA 22.2 No. 61010, 2nd Ed.; UL 61010-1, 2nd Ed.;

DIN EN 61010-1; DIN EN 61010-2-030

Duty Cycle: Continuous

Oven Temperature Range: Ambient to 210° F +/-20%

Timer: 0-30 minutes

SPECIFICATIONS - *(continued)*

Operating Conditions:

Operating Temperatures: 0° C (32° F) to 50° C (122° F)

Relative Humidity: 10% to 90%

Atmospheric Pressure: 650 mmhg to 790 mmhg

Symbol Definitions:

Refer to Exos Oven operator's manual for usage instructions.

Refer to Exos Oven operator's manual for dangers, precautions, and warnings.

DJO, LLC WARRANTIES

The infrared heating system including power head, dome, heating rack, liner pan, base, and all electrical components to be free from defects and workmanship under normal use, when operated in accordance with the above written instructions provided with each unit for **one (1)** year from date of purchase. **DJO, LLC** will repair or replace, at our discretion, any part of the cooking system.

THIS WARRANTY DOES NOT COVER

This Limited Warranty is voided if repairs are made by an unauthorized dealer or the serial number data plate is removed or defaced. Normal deterioration of finish due to use or exposure is not covered by this Warranty. This Limited Warranty does not cover failure, damages or inadequate performance due to accident, acts of God (such as lightning), fluctuations in electric power, alterations, abuse, misuse, misapplications, corrosive type atmospheres, improper installation, failure to operate in accordance with the DJO, LLC's written instructions, or abnormal use.

DJO, LLC'S OBLIGATION

DJO, LLC's obligation under this Limited Warranty is limited to repairing or replacing any part of the infrared cooking system expressly covered by this Limited Warranty which upon examination is found to be defective under normal use. The Limited Warranty is applicable only within the continental United States and only to the original purchaser.

THE LIMITED WARRANTY MAY NOT BE ALTERED, VARIED OR EXTENDED EXCEPT BY A WRITTEN INSTRUMENT EXECUTED BY **DJO, LLC**. THE REMEDY OF REPAIR OR REPLACEMENT AS PROVIDED UNDER THIS LIMITED WARRANTY IS EXCLUSIVE. IN NO EVENT SHALL **DJO, LLC** BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES TO ANY PERSON, WHETHER OR NOT OCCASIONED BY

DJO, LLC's OBLIGATION - *(continued)*

NEGLIGENCE OF THE MANUFACTURER, INCLUDING WITHOUT LIMITATION, DAMAGES FOR LOSS OF USE, COSTS OF SUBSTITUTION, PROPERTY DAMAGE, OR OTHER MONEY LOSS.

Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation exclusions may not apply where such laws apply. This Limited Warranty gives specific legal rights, and there may also be other rights which vary from state to state.

EXCEPT AS OTHERWISE EXPRESSLY PROVIDED ABOVE, **DJO, LLC** MAKES NO WARRANTIES EXPRESSED OR IMPLIED ARISING BY LAW OR OTHERWISE, INCLUDING WITHOUT LIMITATION, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.

RoHS RESTRICTED MATERIALS

- Lead (Pb) 0.1% by weight
- Cadmium (Cd) 0.1% by weight
- Polybrominated Biphenyls (PBB) 0.1% by weight

MATERIAL & TOXICOLOGICAL PROFILE MAXIMUM CONCENTRATION

- Mercury (Hg) 0.1% by weight
- Hexavalent Chromium (Cr-VI) 0.1% by weight
- Polybrominated Diphenyl Ethers (PBDE) 0.1% by weight.
- The EXOS Oven is 100% RoHS compliant! RoHS stands for "the Restriction of the use of certain Hazardous Substances in electrical and electronic equipment". The RoHS Directive took effect on July 1, 2006, and it bans the placing on the European market of new electrical and electronic equipment containing more than the allowed levels of 6 types of potentially harmful heavy metals and chemicals including lead, cadmium, mercury, hexavalent chromium, polybrominated biphenyl (PBB), and polybrominated diphenyl ether (PBDE) flame retardants.

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©2013 DJO, LLC
DJO, LLC | A DJO Global Company
1430 Decision Street | Vista, CA 92081-8553 | U.S.A.