## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

**INTENDED USE/INDICATIONS:** Patients with symptoms, physical findings or radiographic evidence of unicompartimental degenerative knee disease. Brace used to provide pain relief from aforementioned symptoms.

**KONTRAINDICATIONS:** None

**WARNINGS AND PRECAUTIONS:** The brace was designed to complement the variety of medical treatments common to the above afflictions and should be used under the supervision of a medical professional who will determine the frequency and duration of use. If you experience any pain, swelling, sensation changes, circulation problems, or any unusual reactions while using the brace consult a medical professional immediately. The brace should be worn directly against dry skin. The brace is intended for use on a single patient and is not intended for use in contact sports. Braces should not be worn on both right and left legs during activities that may cause the braces to contact each other resulting in potential loss of control and injury. Consult a medical professional for your activity limitations. While the OA Nano, OA Adjuster and 4Titude OA Nano have been designed for function, strength, durability, and comfort, there is no guarantee that injury will be prevented through the use of the brace.

### APPLICATION INFORMATION:

**PRE-APPLICATION** (performed only at initial fitting): With the patient standing, apply the brace varus to the leg without applying the straps. Adjust the brace's varus / valgus alignment to fit the patient's leg contours by adjusting the screws located immediately above and below the hinge with the enclosed Allen wrench (Fig. 1). The screws will flex loose when a neutral brace position is achieved. Continue to adjust the screws to achieve the desired load.

### APPLICATION:

- While seated at the edge of a chair, flex knee to 45° and slip on brace (Fig. 2).
- Center the hinges 1 inch (2.5 cm) above the top of the knee cap and slightly behind the midline of the leg and attach straps #1 and #2 (Fig. 3).
- Push both hinges towards the back of the leg and then attach strap #3 (Fig. 4)
- Draw the hinges forward slightly, but not beyond the midline of the leg, and attach the anti-migration band. Then attach straps #4 and #5 (Fig. 5). For OA Nano and 4Titude OA Nano, anti-migration lock is not to be applied, located in accessory bag; see anti-migration band application instructions.
- The hinges should center just above the top of the knee cap and slightly behind the midline of the leg (Fig. 6).
- Once the brace is on, perform knee bends and walk around to verify no brace slippage occurs (Fig. 7). If necessary, tighten the straps in the numbered order. The strap lengths can be adjusted beneath the strap pad (Fig. 8).

**STRAP PAD APPLICATION (FOR OA NANO ONLY):** For additional comfort, five (5) strap pads are included in the accessory bag. These are placed internally on each strap (via Velcro attachment) to enhance overall fit and comfort.

**ANTI-MIGRATION BAND APPLICATION:** The anti-migration band is used to enhance brace suspension (if needed). Separate application instructions are located in the accessory bag or the OA Nano and 4Titude OA Nano users. The anti-migration band is already attached on all OA Adjuster versions. If removed, please reference the application instructions to reapply.

**HINGE STOP INSTALLATION INSTRUCTIONS:** Detach both condyle pads. Insert 10" extension stops into the front of both hinges. Insert and tighten supplied screws into both hinges (Fig. 9). Re-attach both condyle pads.

**CLEANING INSTRUCTIONS:** Regular Cleaning of components is recommended. A dry lubricant such a Teflon™ spray may be applied to the hinge's internal mechanism and is recommended following use in water. Inspect the hinge screws periodically and tighten if loose. To clean liners and condyle pads, hand wash in 30°C / 85°F cold water using mild soap and use a soft brush. Residual soap may cause irritation and deteriorate material.

**WARRANTY:** DJO, LLC will repair or replace all or part of the product and its accessories for material or workmanship defects for a period of six months for softgoods and one year on brace frames/hinges from the date of sale.

**MATERIAL CONTENTS:** Frame - Magnesium Loading Mechanism - Aluminum & Steel Hinge - Aluminum & Steel Strap tabs - Aluminum & Nylon Soft Goods - Nylon & EVA

Intended for use on a single patient.

**Rx Only**

**NOTICE:** WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUE TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

**CAUTION:** FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTHCARE PROFESSIONAL.

**NOTICE:** WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUE TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

**CAUTION:** FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTHCARE PROFESSIONAL.

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