

(01ES-XS, 01ES-S, 01ES-M, 01ES-L, 01ES-XL)

Aircast® AirSelect™ Short

AirSelect Short:

- For moderate level of support for foot injuries.
- Specifically designed for metatarsal fracture, forefoot and mid-foot injury, acute or post-operative use, bunionectomy, and soft tissue injury.
- Pre-inflated front panel aircell with two customizable distal aircells.

Application Information


A cotton or cast sock will enhance comfort – NOT INCLUDED

1 Prepare Brace


- Unfasten straps.
- Remove front panel.
- Open liner.

2 Apply brace

- While seated, place foot in brace.
- Position heel against back of the brace.
- Wrap liner around foot and then ankle.

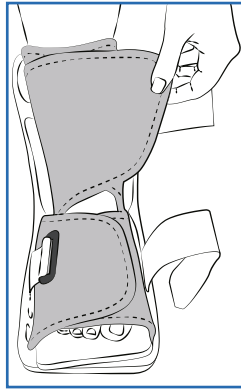


- Replace front panel.
- Position front panel so the Aircast logo is over the toes.
- Secure straps from bottom to top.
- Tighten until snug and comfortable.

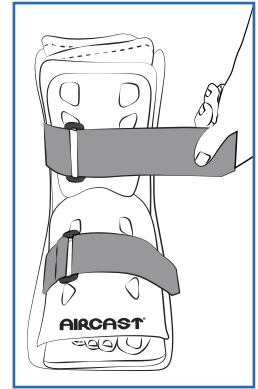

3 Adjust Aircell Compression –


The SHORT walking brace aircells are initially deflated

- Begin inflation on injured side first.
- To inflate aircell, press and release LARGE inflation pump found on the top right-side of brace. Pump until aircell feels snug.
- Repeat to inflate other aircell.

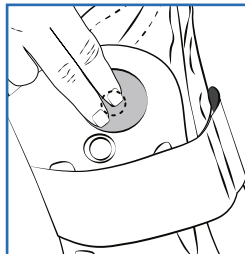

4 Deflate Aircells –

- To deflate aircells, Press and hold finger on SMALL release button.
- Deflate until pressure is comfortable.

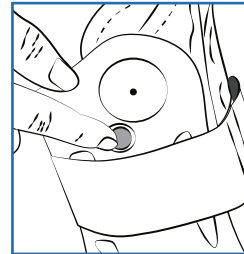

NOTE

Deflate aircells before removing brace for proper fit when reapplying.

Higher pressure provides more support when walking, lower pressure is more comfortable when sitting or reclining. Pressure can also be adjusted by tightening or loosening straps.

CLEANING INSTRUCTIONS:

The foam liner may be hand-washed in 86°F/30°C water with mild detergent and air-dried. Do not put in dryer or use other heat source to dry.

MATERIAL COMPONENTS

Shell: Polypropylene
 Front Panel: High Density Polyethylene
 Straps: Nylon
 Inflation Pumps: Thermoplastic Polyurethane
 Liner: Nylon/Polyurethane/Polyester (Outer/Foam /Inner Mesh)
 Binding: Polyester Outer Sole / Insole: Eva/Rubber

PATENT PENDING

DJO, LLC.
 1430 Decision Street
 Vista, CA 92081-8553 U.S.A.
 Phone 760.727.1280
 Toll Free 800.526.8785
 Fax 800.457.4221
djoglobal.com/aircast



DJOGlobal.com