


DonJoy Back Brace II TLSO

PATIENT INSTRUCTIONS



Before using the device, please read the following instructions completely and carefully. Correct application is vital to the proper functioning of the device.


Application of Brace:

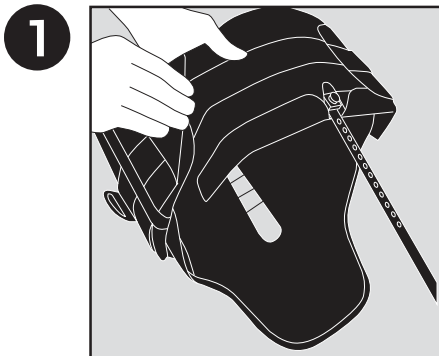

1

Open The Brace.


Prior to applying, open one of the Velcro side panels. Keep the shoulder straps attached.

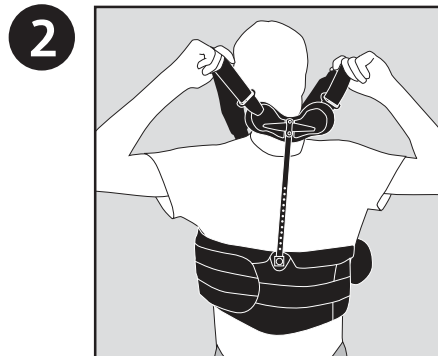

2

Apply The Brace.


While standing, lift the brace over the head and lower the brace into position. Let the front panel and back panel hang over the torso.

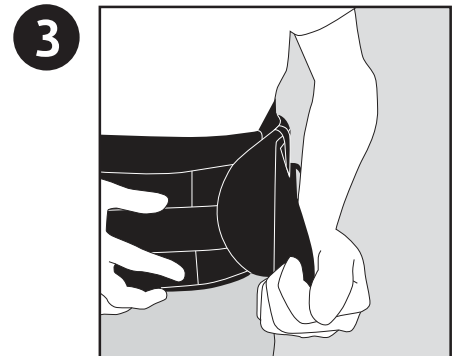

3

Close The Brace.

Close both of the Velcro side panels by overlapping Velcro side panels onto front panel. Ensure that Velcro side panels are even and comfortably snug. Readjust shoulder straps if needed.

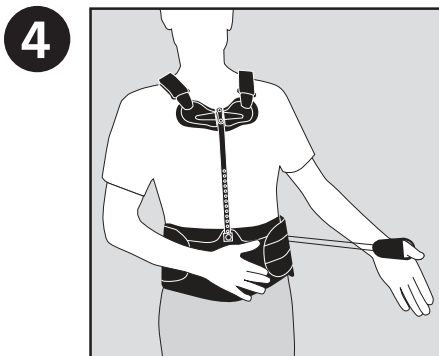

4

Activate compression.

Grasp the pull handle and pull straight out from the torso. Fasten the pull handle to front panel. Adjust compression as needed.

Removal of Brace:

Remove the pull handle from the front panel and release tension. Attach the pull handle to the side panel of the brace. Open one of the Velcro side panels. Unbuckle the shoulder strap on the same side as the open Velcro panel. Slide the brace off the other shoulder like removing a jacket or backpack.

Washing the Brace:

Remove the rigid front and back panels, and hand wash the brace in cold water (not to exceed 86° F) with mild soap. Allow the brace to air dry.

Patient Education Video:

Additional information and a patient education video are available at the DJO Global web site, or may be accessed using this QR icon.

<http://www.djoglobal.com/products/donjoy/donjoy-back-brace-ii>

MATERIALS: BACK PANEL: Aluminum, Polyester, Polyurethane Foam, Nylon, Polypropylene; **WAIST BELT:** Polyester, Polyurethane Foam, Nylon, Polypropylene; **SHOULDER STRAPS:** Nylon, Polyester, Polyurethane Foam, Polypropylene.

PRODUCT NOT MADE WITH NATURAL RUBBER LATEX. SINGLE PATIENT USE ONLY

CONTRAINDICATIONS: Pregnancy and circulation, pulmonary, cardiovascular or skeletal conditions which may worsen as a result of compression of the abdomen and/or pressure.

RX Only



For product assistance, please contact Product Support at **1-888-405-3251** or email product.specialist@djoglobal.com