


Spine Bracing System PATIENT APPLICATION INSTRUCTIONS

## EXOS FORM<sup>™</sup> II 621

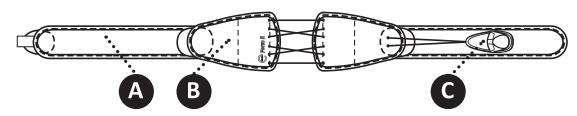

## **Application of Brace:**

APPLY BRACE by centering Rear Panel Section just above buttock crease. Wrap Belt Wings (A) in front

about 1" (2.5 cm) above

secure hook and loop.



LOOSEN BRACE by

detaching the Pull Tab (C) and pull Rear Panel Section (B) in opposite directions.



TIGHTEN BRACE by grasping Pull Tab and pulling away from body. Secure Pull Tab to Belt Wing.



**BRACE REMOVAL:** Detach Pull Tab from Belt Wings so tension becomes loose. Detach the hook and loop connection on Belt Wings and remove belt.

## **USE AND CARE (CLEANING & STORAGE**

**INSTRUCTIONS):** Hand wash in cold water, less than 86°F (30°C), using mild soap. Air dry. NOTE: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material. Do NOT iron. Do NOT expose to heat over 120°F (49°C). Do NOT tumble dry. Do NOT bleach.



For more information including application videos, please visit DJOGlobal.com/products/exos/exos-form-ii-back-brace or call 888.405.3251

MATERIAL CONTENTS: Nylon, Polyethylene, Polyester, Antimicrobial, Polyurethane, Polyvinylchloride, Silicone, Elastic. FOR SINGLE PATIENT USE ONLY. NOT MADE WITH NATURAL RUBBER LATEX.

CONTRAINDICATIONS: Pulmonary, cardiovascular or skeletal conditions which have risk to be made worse as a result of compression and/or pressure; unstable, displaced fractures

For product assistance, please contact Product Support at 1.888.405.3251 or email product.specialist@djoglobal.com

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

Next Appointment: Date:\_ Time:

DJO, LLC | A DJO Global Company T 800.336.6569 D 760.727.1280 F 800.936.6569 1430 Decision Street I Vista, CA 92081-8553 I U.S.A.

DJOglobal.com/exos



U.S.A. and International Patents Pending ©2016 DIO. LLC 00-4185 Rev A

