

Bucks Traction

Patient Application Instructions





APPLICATION INSTRUCTIONS FOR THE FOLLOWING BUCKS TRACTION DEVICES:

Full Foam Bucks Traction Universal Bucks Traction with Foot Pad

BEFORE USING THE DEVICE, PLEASE READ THE **INSTRUCTIONS FOR USE** COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

- **1a)** Unfasten or loosen straps. Elevate the patient's leg and slide the splint underneath.
- **1b)** Seat the heel as far back in the pocket as possible. (If your splint has a separate forefoot pad, to help impede footdrop, then slip the foot between the pad and the splint.)
- **1c)** Tighten and secure straps. (Straps may need to be readjusted after traction weight is applied.)
- 2) Check regularly for good alignment and good circulation.

CLEANING INSTRUCTIONS:

Hand wash liner in warm water using mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

WARNINGS AND PRECAUTIONS:

If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. Never wear support over an open wound.

MATERIAL CONTENTS:

Nylon, Polyester, Spandex, Polyurethane NOT MADE WITH NATURAL RUBBER LATEX.

INTENDED USE/INDICATIONS:

To be used in conjunction with leg traction for comfort and immobilization.

CONTRAINDICATIONS: None

WARRANTY:

DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

FOR SINGLE PATIENT USE ONLY.

Individual results may vary. Neither DJO, LLC nor any of its subsidiaries dispense medical advice. The contents of this document do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.



 DJO, LLC
 A DJO Global Company

 T
 800.336.6569
 D
 760.727.1280
 F
 800.936.6569

 2900 Lake Vista Drive
 I
 Lewisville, TX
 75067
 I
 U.S.A.

 DJOGlobal.com
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D
 D