

CLEANING AND STORING

DonJoy[®] IceMan[®] Therapy Units help you manage pain and swelling with the controlled delivery of cold therapy. Here's how to keep your unit in top condition with cleaning and storing and keep you on the road to recovery.

CLEANING AND STORING

- With dry hands, unplug the power cord from the back of the unit and the outlet
- 2. Empty the water/ice from cooler
- 3. Remove filter cap for cleaning
 - a. Filter cap is located at the bottom of the mixing chamber, inside the unit
 - b. Clean the filter cap then re-attach
- Refill cooler with warm water to the fill line and add 1-2 tablespoons of bleach
- 5. Replace the lid, place unit on a dry level surface, connect the pad, and plug in
 - a. Let the unit run for 10 minutes
- 6. Empty the cooler and wipe the inside of the unit dry
 - a. Leave the lid off the unit for at least 1 day to air dru
- Drain cold pad with the hose facing downward by pushing the ends of the connectors
 - a. Some residual water may remain in the pad.



Scan the QR code provided for a video on cleaning and storing your DonJoy IceMan unit

enovis.

T 800.336.6569 F 800.457.4221

DJO, LLC 5919 Sea Otter Place, Suite 200 | Carlsbad, CA 92010 | U.S.A. enovis.com/doniou

Copyright © 2022 DJO, LLC MKT00-12444 RevA

Warning content: This device can be cold enough to cause serious injury. DO NOT use this device without a prescription from a physician.

Individual results may vary. Neither DJO, LLC nor any of the Enovis companies dispense medical advice. The contents of this document do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.