## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

**INTENDED USE/INDICATIONS:** The Aircast SP Walker is designed to provide support and immobilization to the ankle. It may be suitable for the treatment of stable fractures of the foot, lateral ankle sprains and ligamentous injuries such as plantar fasciitis. Providing immobilization or controlled movement of the limb or body segment.

**CONTRAINDICATIONS:** This device is contraindicated for unstable fractures of the lower leg, ankle and foot.

### WARNINGS AND PRECAUTIONS:

- This device is to be used under the supervision of a healthcare professional. The determination of when to apply the brace and the frequency and duration of use should be strictly at the discretion of the treating physician. Always consult with your physician before making changes to the brace.
- Do not use this device on patient's incapable of communicating physical discomfort.
- Do not over-inflate aircells. Improper aircell inflation may cause significant skin irritation, especially in people with diabetic neuropathy. Reduce aircell pressure with any visual skin changes or reported discomfort.
- Patients w/without sensation (i.e. post-op anesthesia, neuropathies, etc.) should be monitored frequently for skin damage, skin irritation or wound management.
- Use caution when walking on slippery or wet surfaces to avoid injury.
- Do not over tighten straps. This may result in reduction of blood flow or sensation.
- Do not use over open wounds.
- Do not modify the device or use the device other than intended.
- At high altitudes the aircells will expand beyond their optimal level. Adjust aircell support by tightening or loosening straps.
- If you develop an allergic reaction and/or experience itchy, red skin after contact with any part of this device, please stop using it and contact your healthcare professional immediately.
- Do not use this device if it is damaged and/or the packaging has been opened.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, consult your medical professional immediately.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

**APPLICATION INFORMATION:** (A cotton or cast sock will enhance comfort – NOT INCLUDED)

- PREPARE BRACE:** Unfasten straps, remove front panel, and open liner.
- APPLY BRACE:** While seated, place foot in brace. Position heel in back of brace and pull liner forward to remove any wrinkles (Fig. 1). Wrap liner around foot, then ankle, smooth wrinkles, and secure (Fig. 2). Position front panel so that Aircast logo is over the toes. Secure front panel with lower part inside brace, and upper part outside brace. Secure straps from bottom to top (Fig. 3). Tighten until snug and comfortable.
- ADJUST AIRCELL SUPPORT:** SP-Walker aircells are initially deflated. To inflate aircells, firmly insert the "inflate" tip of hand bulb into valve and squeeze until aircell feels snug (Fig. 4). Begin with valve on inside of leg, then valve on outside of leg.

In the case of over-inflation, deflate aircells. To deflate aircells, insert the "deflate" tip of hand bulb into valve and squeeze until pressure is comfortable.

Higher pressure gives more support when walking, lower pressure is more comfortable when sitting or reclining. (Pressure can also be adjusted by tightening or loosening straps.)


**USE AND CARE:** The SP Walker foam liner may be hand-washed in cold water with mild detergent and air-dried. Do not put in dryer or use other heat source to dry.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

### INTENDED FOR SINGLE PATIENT USE.

~~NOT MADE WITH NATURAL RUBBER LATEX.~~

~~RX ONLY.~~

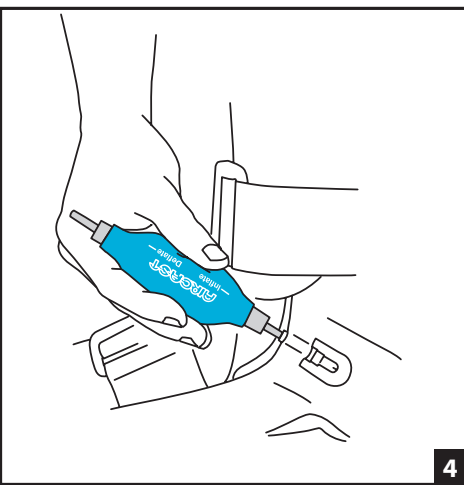**NOTICE:** WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNOLOGY TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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