ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS: The Aircast XP Walker (Extra Pneumatic) provides support, and immobilization to the lower leg, ankle and foot. It may be suitable for use following stable ankle and/or foot fractures and severe lateral ankle sprains. Providing immobilization or controlled movement of the limb or body segment.

CONTRAINDICATIONS: This device is contraindicated for unstable fractures of the lower leg, ankle and foot.

WARNINGS AND PRECAUTIONS:

- This device is to be used under the supervision of a healthcare professional... Do not use this device on patients' incapable of communicating physical discomfort... Do not over-inflate aircells. Improper aircell inflation may cause or worsen skin irritation... Patient's without sensation (i.e. post-op anesthesia, neuropathies, etc.) should be monitored frequently for skin damage, skin irritation or wound management... If pain, swelling, changes in sensation or other unusual reactions occur while using this product, consult your medical professional immediately.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION:

- 1. PREPARE BRACE: Unfasten straps, remove front panel, and open toe cover and foot flaps. 2. APPLY BRACE: Put on sock included with brace. While seated, place leg in brace and seat heel against back. Wrap foot flaps over foot and toe cover around toes (Fig. 1). 3. ADJUST AIRCELL SUPPORT: Pneumatic/Walker aircells are initially deflated. To inflate, firmly insert "inflate" tip of hand bulb into brace valve. 4. ADJUST DEFLATION TIP: In the case of overinflation, firmly insert "deflate" tip of hand bulb into brace valve and squeeze several times until desired compression is reached.

NOTE: The brace is more comfortable when pressure is higher in distal aircells than proximal. Higher pressure in aircells provides more support when walking. Lower pressure is more comfortable when sitting or reclining.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

USE AND CARE: The foam liner may be hand-washed in cold water with mild detergent and air-dried.

NOT MADE WITH NATURAL RUBBER LATEX. INTENDED FOR SINGLE PATIENT USE. RX ONLY.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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