

Document No. 0400-0259	Rev. A	Pg. 1 of 2	Approvals / Date
Title: INSTRUCTIONS FOR USE – DJO ADDENDUM FOR EXPRT PRECISION SYSTEM: REVISION KNEE, STERILE FEMORAL AUGMENT			D. RECTOR
			C. ARCHER/M. COPPO
			J. VEGA-HUSSEY
			T. HUTTO
			L. SILVERMAN

Revision	ECO Date	ECO	Summary of Changes
A	OCT-2016	DC-06568	RELEASE TO MANUFACTURING AT REVISION "A".

ADDENDUM TO IFU 0400-0034

0400-0259 Rev. A 2016-10

EN

SECTION 2. Product Description and Implant Materials

Component	Fixation Method	Material	Applicable ASTM Standard	Applicable ISO Standard
Exprt™ Revision Femoral Augment	Cemented	Ti6Al4V alloy	ASTM F136 / ASTM F1472	ISO 5832-3

1. MRI Safety

DJO Surgical Knee components have not been reviewed for safety and compatibility in the MR environment. The devices have not been tested for heating or migration in the MR environment. The risks associated with a passive implant in the MR environment have been evaluated and are known to include heating, migration, and image artifacts at or near the implant site.