ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS

Elastic wrist support with malleable aluminium palmar stay and removable compression strap

FEATURES

- Removable, malleable palmar aluminum stay for stabilization of the wrist
- Removable strap to adjust compression level
- 3-D anatomically contoured knit provides comfortable, uniform compression
- Breathable material

INDICATIONS

- Chronic, post-traumatic or postoperative inflammation of the wrist joint and/or soft tissues
- Wrist sprains
- Wrist Tendinitis (tenosynovitis)
- Mild Osteoarthritis
- Rheumatic wrist pain (arthritis)
- Carpal Tunnel Syndrome

CONTRAINDICATIONS

The support should only be used after consulting a health care professional if you suffer from the following symptoms:

- Skin diseases and injuries in the injured area (e.g. allergy-related skin changes, irritations, scars with swelling, redness, hyperthermia)
- Lymphatic disorders, indistinct soft tissue swelling on affected area
- Circulatory or sensory disorders in affected area

WARNING & PRECAUTIONS

If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.

If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.

APPLICATION INSTRUCTIONS

- Choose the support in left or right model according to the affected hand.
- Pull the support over your hand and wrist, placing the thumb in the opening on the side.
- For correct positioning, ensure that the hole of the silicone insert located on the interior of the support, covers the tuberosity on the lateral (external) part of the wrist.
- The removable palmar aluminum stay can be adjusted to individual needs by bending it carefully.
- Freely, attach the removable strap to the small velours piece on the internal side of the support and wrap it around the wrist. The strap will provide additional compression and stability, adjust it according to your needs. Avoid tightening the strap too much and loosen it immediately if you feel it is too tight.

Advice: When using the removable strap, avoid direct contact between the Hook & Loop of the strap and the knitted support, as this will damage the visual aspect of the knit.

If you no longer need the aluminum stay for stabilization it can be removed. To remove the stay, hold the support by the top end and gently pull the aluminum stay out of its pocket.

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USAGE INSTRUCTIONS

- Worn directly on the skin, the skin should be free from injuries, moisture, oils, creams, gels or similar residues when support is being used.
- The support should not be worn for more than 8 hours a day.
- With proper use and the observance of all instructions, no pressure points should occur.
- The manufacturer shall bear no responsibility for any changes to the product, or incorrect usage.

CARE

- Hand wash in lukewarm water (30 °C) with a mild detergent
- Air dry (do not place on radiators/heaters or other heat sources)
- Do not use bleach or chlorine do not iron.

COMPOSITION

- Knitted material: Polyamid, elastin, polyester, natural rubber latex
- Insert: silicone
- Internal Cover: tery cloth
- Strap: Polyamid
- Stay: Aluminium

LATEX

THIS PRODUCT CONTAINS NATURAL RUBBER LATEX.

FOR SINGLE PATIENT USE ONLY

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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