## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### INDICATIONS:

The Procare Elastic Ankle Support/Double Strap Ankle Support is designed to support the ankle. It may be suitable for use following mild ankle sprains and strains and soft-tissue injuries. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

### CONTRAINDICATIONS: N/A

### APPLICATION INSTRUCTIONS:

- Product should be worn over an athletic sock.  
A) Grasp top edge of support and roll over to create a short cuff. (Fig. 1)  
B) Pull support onto the ankle as shown. (Fig. 2)

### For Double Strap Ankle:

- C) Wrap each strap in opposite direction across the top of the foot, around ankle, and affix hook end to sewn down loop material. (Fig. 3)

### CARE INSTRUCTIONS:

Hand wash in cold water with mild soap, rinse thoroughly, AIR DRY.  
**Note:** If not rinsed thoroughly, residual soap may cause irritation or deteriorate material.

### WARNINGS AND PRECAUTIONS:

- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
  - Do not use over open wounds.
  - Do not use this device if it was damaged and/or package has been opened.
  - If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.
- NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### FOR SINGLE PATIENT USE ONLY.

NOT MADE WITH NATURAL RUBBER LATEX.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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