ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to: Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS:

The Procure Posterior Tibia/Fibular splint and Femoral Leg Splint are designed to provide support and immobilization to the lower leg, ankle and foot. It may be suitable for use following a knee injury, soft tissue injury, ankle fracture or lateral ankle sprain. Providing immobilization or controlled movement of the limb or body segment.

CONTRAINDICATIONS: None.

WARNINGS AND PRECAUTIONS:

- It is recommended to have the device fitted by a suitably qualified health care professional.
 - If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.
 - If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
 - Do not use over open wounds.
 - Do not use this device if it was damaged and/or packaging has been opened.
- NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION:

A) Place splint on the leg in the desired position and secure with an elastic wrap, wrapping distal to proximal.

USE AND CARE:

- The splint is malleable and can be molded if desired.
- The splint is autoclavable.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

FOR SINGLE PATIENT USE ONLY.

 NOT MADE WITH NATURAL RUBBER LATEX.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2020 DJO, LLC