1

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to: Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS: The Procure Squared-Toe Post-Op shoe is designed to provide support and protection to the foot. It may be suitable for Use following surgery or trauma to the foot. Providing mild protection of a limb or body segment for acute and prophylactic care.

PERFORMANCE CHARACTERISTICS:

Providing mild protection of a limb or body segment for acute and prophylactic care.

CONTRAINDICATIONS: None.

WARNINGS AND PRECAUTIONS:

This product is to be used under the supervision of a medical professional namely a physician and not intended for unsupervised public use.

- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
Do not use over open wounds.
Do not use this device if it was damaged and/ or packaging has been opened. If you develop an allergic reaction and/ or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS:

- Unfasten ankle strap and forefoot closure. Slip foot into the shoe.
Secure both strap and forefoot closure for a snug but comfortable fit.

CLEANING INSTRUCTIONS:

To clean shoe, wipe with a damp cloth and air dry.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

FOR SINGLE PATIENT USE ONLY

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.

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©2020 DJO, LLC