## ITALIANO

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13-4296-0-00000 REV C - 2020-12-16

## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to: Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

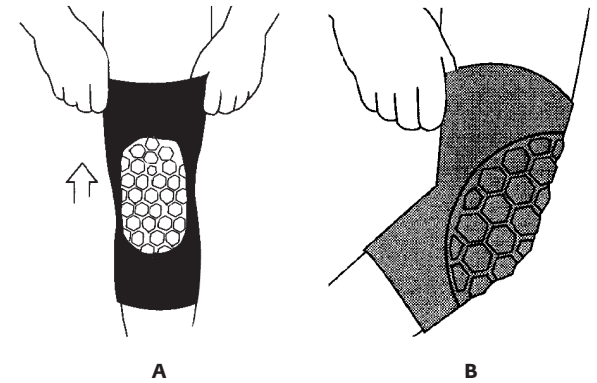
**INTENDED USE/INDICATIONS:** The DonJoy Spider Pad is designed to provide protection to the knee on impact. For impact protection. Providing mild protection of a limb or body segment for acute and prophylactic care.

**CONTRAINDICATIONS:** None.

### APPLICATION INFORMATION:

**A) SPIDER PAD KNEE:** Grasp larger opening and pull over knee so that the padded area covers the knee cap.

**B) SPIDER PAD ELBOW:** Grasp larger opening of the sleeve, pull on to the elbow so that padded area covers the elbow.

**CLEANING INSTRUCTIONS:** Hand wash in cold water with mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause skin irritation and deterioration of the material.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

**WARNINGS AND PRECAUTIONS:** It is recommended to have the device fitted by a suitably qualified healthcare professional. If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately. If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately. Do not use over open wounds. Do not use this device if it was damaged and/or packaging has been opened.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device

FOR SINGLE PATIENT USE ONLY.  NOT MADE WITH NATURAL RUBBER LATEX.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2020 DJO, LLC