## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

**INTENDED USER PROFILE:** The intended user should be a licensed medical professional, the patient, the patient’s caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

**INDICATIONS:** Patellar Tendinitis, Osgood-Schlatter’s Disease or Chondromalacia Patella. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

A) With the leg slightly flexed, position the wide part of the strap with the DonJoy logo at the back of the knee.

B) Wrap the straps around the front of the knee and insert the ends through the slits in the straps so it can be tightened. Snug to comfort. Position should be just under the patella.

**CONTRAINDICATIONS:** Do not use if you are allergic to any of the materials contained in this product.

**USE AND CARE:** Hand wash in cold water with mild detergent. Air dry.

### WARNINGS AND PRECAUTIONS:

- Do not use if you have varicose veins.
- Do not wear to bed or in an AirPlane.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or package has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device

**MATERIAL CONTENT:** Neoprene

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

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©2020 DJO, LLC