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13-5252-0-00000 REV D 2020-12-14

APPLICATION INSTRUCTIONS

Before applying the brace, please read these instructions completely and carefully.
Correct application is vital to the proper functioning of the brace.

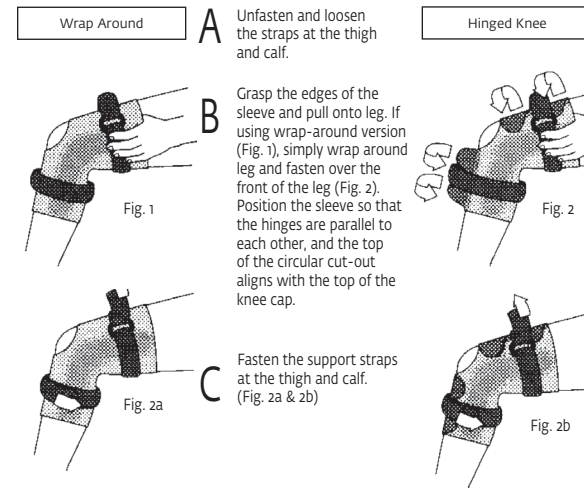
INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INDICATIONS:

The DonJoy Economy Hinged Brace is designed to support the knee. It may be suitable for those suffering from mild to moderate MCL and/or LCL instabilities, mild global instabilities and soft-tissue injuries. Soft-good/semi-rigid designed to restrict motion through elastic or semi rigid construction.

CONTRAINDICATIONS: N/A

D REMOVAL
To remove, unfasten straps and apply a downward pressure to the top of the sleeve at the hinges.

USE AND CARE

- Hand wash in cold water using mild soap, rinse thoroughly. AIR DRY. Hinges may be removed or washing. Note: If not rinsed thoroughly, residual soap may cause skin irritation and deterioration of the material.
- The brace may be used in fresh or salt water. Following use in water activities, drain thoroughly and rinse with clean, fresh water. AIR DRY.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.
WARNING: NEOPRENE SUPPORTS SHOULD NOT BE WORN BY INDIVIDUALS WITH KNOWN SUSCEPTIBILITY TO DERMATITIS OR ALLERGIES TO NEOPRENE BY-PRODUCTS.

WARNINGS AND PRECAUTIONS:

NEOPRENE SUPPORTS SHOULD NOT BE WORN BY INDIVIDUALS WITH KNOWN SUSCEPTIBILITY TO DERMATITIS OR ALLERGIES TO NEOPRENE BY-PRODUCTS.
• Do not use over open wounds. • Do not use this device if it was damaged and/or package has been opened. • If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately. • If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately. Please be advised that DJ Orthopedics, LLC, does not recommend simultaneous wearing of functional knee braces bilaterally (right and left legs) during activities (i.e. skiing) that may cause the braces to contact each other, resulting in potential loss of control and injury. Functional knee braces are not recommended to be worn over clothing. Consult your licensed healthcare professional for your activity limitations. NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

CAUTION: Please be advised that DJ Orthopedics, LLC, does not recommend simultaneous wearing of functional knee braces bilaterally (right and left legs), during activities (i.e. skiing) that may cause the braces to contact each other, resulting in potential loss of control and injury. Functional knee braces are not recommended to be worn over clothing. Consult your licensed healthcare professional for your activity limitations.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2020 DJO, LLC