## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

**INTENDED USE/INDICATIONS:** Provides mild support to the knee.

**CONTRAINDICATIONS:** N/A

### WARNINGS AND PRECAUTIONS:

- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or package has been opened.
- If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### APPLICATION INSTRUCTIONS:

A) Grasp edges of sleeve at the larger opening. Pull the sleeve onto the knee. (For elastic knee support, roll a small cuff in the larger opening to prevent stress in the material). (Fig. 1 & 2)

B) For sleeves with donuts or patella cutouts, position the sleeve so that the donut or cut-out is centered on the knee cap. (Fig. 3 & 4)

**CLEANING INSTRUCTIONS:** Hand wash in cold water with mild soap, rinse thoroughly. AIR DRY. DO NOT MACHINE DRY or bleach.

**NOTE:** If not rinsed thoroughly, residual soap may cause skin irritation and deterioration to the material.

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