

Description

CellerateRX® Surgical Activated Collagen® Powder is Type I bovine hydrolyzed collagen and contains no additives.

Indications

For use in the management of acute and chronic wounds, including surgical wounds, partial- and full-thickness wounds, traumatic wounds, venous stasis ulcers, arterial ulcers, diabetic ulcers, and first- and second-degree burns.

When to use CellerateRX Powder

Use CellerateRX powder on surgical wounds and moderately to heavily exudative wounds.

Initial Application

Cleanse the wound per facility protocol. Apply CellerateRX[®] Powder over the entire wound bed and the edges of the wound. Cover with an appropriate dressing. Infected wounds are treated per physician direction.

Reapplication

Reapply CellerateRX Powder 2-3 times per week or with each dressing change, taking care not to disrupt the wound site.

Contraindications

Known sensitivity to bovine collagen

Single use
Consult instructions for use
STERILE Sterile until opened
Do not use if package is damaged
Store at room temperature

Learn more at crxsurgical.com

Manufactured in the USA for: Wound Care Innovations, LLC, a Sanara MedTech Inc. Company 1200 Summit Avenue, Suite 414 • Fort Worth, TX 76102 • 800-205-7719

CellerateRX® is a trademark of Applied Nutritionals, LLC • Activated Collagen® is a trademark of Wound Care Innovations, LLC, a Sanara MedTech Inc. Company © 2020 Sanara MedTech Inc. All rights reserved. Rev. 190726