ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INDICATIONS: Provides cold or heat therapy after surgery or injury.

INSTRUCCIONES:

FOR COLD THERAPY:

- PREZING:** Place gel inserts in the re-sealable bag. Place re-sealable bag with inserts in freezer until the gel inserts is frozen solid.
- PRE-APPLICATION:** remove the gel inserts from the re-sealable bag. Place the gel insert(s) in Dura®Soft wrap.
- APPLICATION:** Apply wrap to area of treatment by attaching hook or velcro fastener to outer wrap fabric. The straps should be applied with enough stretch to keep wrap in place and provide desired compression.
- ADDITIONAL APPLICATIONS:** when insert(s) no longer deliver desired treatment, remove insert(s) from wrap and exchange gel insert(s) in re-sealable bag. Repeat step 3 Application.
- REUSE:** wrap or inserts may be used again by repeating steps 1 through 4.

FOR HEAT THERAPY:

- HEATING:** Place gel insert(s) in deep pan. Run hot tap water into pan until insert(s) is covered. Keep immersed for 3 – 5 minutes. Do not use tap water which is too hot to the touch.
- PRE-APPLICATION:** Dry the gel insert(s) and place the insert(s) in Dura®Soft wrap.
- APPLICATION:** Apply wrap to area of treatment by attaching hook or velcro fastener to outer wrap fabric. The straps should be applied with enough stretch to keep wrap in place and provide desired compression.
- ADDITIONAL APPLICATIONS:** When the insert(s) no longer deliver desired treatment, remove the insert(s) from wrap, exchange gel insert(s) in re-sealable bag. Repeat steps 1 through 3.
- REUSE:** wrap or inserts may be used again by repeating steps 1 through 4.

CONTRAINDICATIONS:

FOR COLD THERAPY: Sickle-cell Anemia, Raynaud's or other vasospastic diseases, cold allergy, cold hypersensitivity, previous cold injury, or any compromised local circulation, decreased skin sensitivity, hypertension, paroxysmal cold hemoglobinuria, and cryoglobulinemia.

FOR HEAT THERAPY: heat therapy should not be used over areas of dermatitis, recent hemorrhage or potential hemorrhage, inflammation, open wound, infection, malignant tumors. Other contraindications include impaired skin sensation (particularly to temperature and pain), advanced heart disease, peripheral vascular disease, significant hepatic or renal insufficiency, and previous heat injury.

PRECAUTIONS:

- If the effect and duration of cold or heat therapy is unknown, consult a physician before use.
- Any cold products may cause injuries if improperly used.
- The temperature of the hot pack should be comfortably warm. Insert that is too hot to the touch may cause injuries.
- Extreme care must be taken when using any cold or heat therapy, especially for Children, Elderly, diabetics, unconscious or incapacitated patients, those with decreased skin sensitivity, or poor circulation due to cardiovascular diseases, medications, tobacco and alcohol use.
- If redness, numbness, burning, pain, tingling sensation increases, or unusual reaction like burns, blisters, discoloration, swelling, or other reactions in skin appearance occurs while using this product consult a physician immediately.

WARNING:

- Neoprene wrap should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by-products.
- Keep the product away from children.
- All ingredients are non-toxic and are not harmful to skin. They are, however, not intended for ingestion.
- Do not apply the gel inserts directly to the skin.
- Do not wrap the pad so tightly or limit compression as to restrict blood or fluid flow.
- Never lay or sit on top of Dura®Soft gel insert.
- Do not microwave the inserts.

USE AND CARE:

- Neoprene and hand washed and air dried. Gel inserts may be cleaned with water, moist towel or sponge.
- For cold therapy, leave the gel inserts in freezer until required. Provide proper insulation if the gel inserts are removed from freezer more than 1 hour prior to use.
- For heat therapy, store gel inserts in the re-sealable bag until needed.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

FOR SINGLE PATIENT USE ONLY.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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