

FORM FIT™ CERVICAL COLLAR

Clinical Application Instructions




BEFORE USING THIS MEDICAL DEVICE, PLEASE READ THE **INSTRUCTIONS FOR USE** COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE. THIS PRODUCT IS INTENDED TO BE USED UNDER THE SUPERVISION OF A HEALTHCARE PROFESSIONAL.

IDEAL FOR: Post-operative rehabilitation, cervical sprains and strains.

SIZING:

- 1** Measure the height from the sternal notch to the chin on patient. (figure A)
- 2** Next, measure the circumference of the patient's neck. (figure B)
See chart at right for size selection.

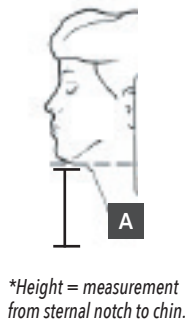


H* X L	CIRCUM.	SIZE
2" x 14"	7" – 12"	Ped./2XS
2" x 16.5"	9" – 13"	XS
3" x 18.5"	11" – 16"	S
3" x 22"	15" – 20"	S/Long
4" x 20"	13" – 18"	M
4" x 23"	16" – 21"	M/Long
4.5" x 22.5"	15" – 20"	L
4.5" x 24"	16" – 22"	XL
2" x 24"	17" – 22"	Long/Nar.



APPLICATION:

- 1** Center the cervical collar under the chin. (Figure 1)
- 2** Wrap both ends around the back of the neck and secure hook and loop closures. (Figure 2)
Adjust for comfort, ensure collar doesn't interfere with normal breathing.

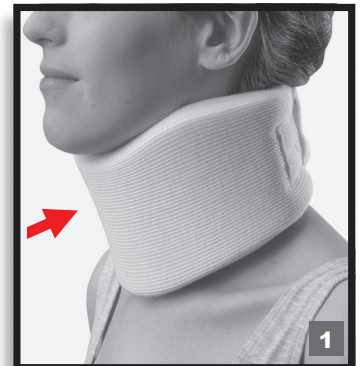

CLEANING INSTRUCTIONS:

Hand wash in cold water using mild detergent. Rinse thoroughly and air dry.

MATERIAL CONTENTS:

NOT MADE WITH NATURAL RUBBER LATEX.

WARRANTY:

DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale

WARNINGS AND PRECAUTIONS:

If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

Rx ONLY.

INTENDED FOR SINGLE PATIENT USE.

For product assistance, please contact Product Support at 1.888.405.3251 or email product.specialist@djoglobal.com

Individual results may vary. Neither DJO, LLC nor any of its subsidiaries dispense medical advice. The contents of this poster do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.



DJO, LLC | A DJO Global Company

T 800.336.6569 D 760.727.1280 F 800.936.6569
2900 Lake Vista Drive | Dallas, TX 75067 | U.S.A.

DJOGlobal.com