
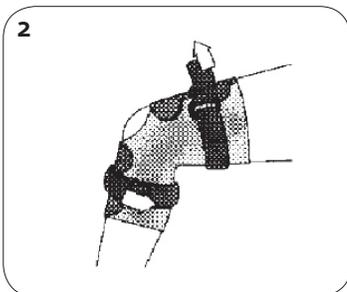


Reddie™ Brace

PATIENT APPLICATION

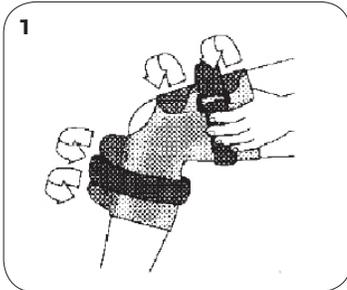

BEFORE USING THIS DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS:

Support for mild MCL and/or LCL sprains · Mild arthritis · Post arthroscopic menisectomy · Mild global instabilities.

WARNINGS AND PRECAUTIONS:

NEOPRENE SUPPORTS SHOULD NOT BE WORN BY INDIVIDUALS WITH KNOWN SUSCEPTIBILITY TO DERMATITIS OR ALLERGIES TO NEOPRENE BY PRODUCTS.

WARNING:

THIS PRODUCT IS TO BE USED UNDER THE SUPERVISION OF A MEDICAL PROFESSIONAL. THIS DEVICE IS NOT INTENDED FOR UNSUPERVISED PUBLIC USE. IF YOU EXPERIENCE ANY PAIN, SWELLING, SENSATION CHANGES, OR ANY UNUSUAL REACTIONS WHILE USING THIS PRODUCT, CONSULT YOUR MEDICAL PROFESSIONAL IMMEDIATELY.

APPLICATION INFORMATION:

- A) Unfasten and loosen the straps at the thigh and calf.
- B) Wrap product around leg and fasten over the front of the leg. Position the sleeves so that the hinges are parallel to each other, and the top of the circular cut-out aligns with the top of the knee cap.
- C) Adjust posterior (back) thigh and calf straps for snug fit.
- D) Fasten the anterior (front) thigh and calf strap.
- E) **REMOVAL** To remove, unfasten straps and front straps.

USE AND CARE:

- Hand wash in cold water using mild soap, rinse thoroughly. AIR DRY. Hinges may be removed for washing.
NOTE: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.
- This brace may be used in fresh or salt water. Following use in water activities, drain thoroughly and rinse with clean, fresh water. AIR DRY.

WARRANTY:

DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

FOR SINGLE PATIENT USE

NOTICE: NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE OF THE ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

CAUTION: FEDERAL LAW (U.S.A.) - RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.



DJO, LLC

T 800.336.6569 D 760.727.1280 F 800.936.6569
1430 Decision Street | Vista, CA 92081-8553 | U.S.A.
DJOglobal.com