

ProCare® Transitional 172 Cervical Collar- Adult Sizing Guide

Part Number	Description	Height	Neck Circumference	Color Code
79-83363	Transitional 172 Collar	4.25"	13"-22"	Adult X-Tall
79-83365	Transitional 172 Collar	3.75"	13"-22"	Adult Tall
79-83367	Transitional 172 Collar	3.0"	13"-22"	Adult Regular
79-83368	Transitional 172 Collar	2.25"	13"-22"	Adult Short
Part Number	Description	Height	Neck Circumference	Color Code
79-83283	Transitional 172 Collar w/replacement pads	4.25"	13"-22"	Adult X-Tall
79-83285	Transitional 172 Collar w/replacement pads	3.75"	13"-22"	Adult Tall
79-83287	Transitional 172 Collar w/replacement pads	3.0"	13"-22"	Adult Regular
79-83288	Transitional 172 Collar w/replacement pads	2.25"	13"-22"	Adult Short

READ INSTRUCTIONS BEFORE USE.

Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.

For More Information Please Contact:


Individual results may vary. Neither DJO, LLC nor any of its subsidiaries dispense medical advice. The contents of this document do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.

PROCARE® **Adult Sizing Guide**
Transitional 172 Cervical Collar

Please refer to the ProCare® Instructions for Use for more information.

Select the Right Size


Position the patient into neutral head alignment, place the guide on the highest point of the trapezius and flat against the side of the head.

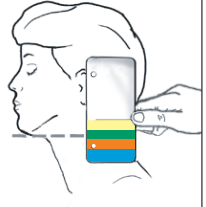

Visualize an imaginary horizontal line from the bottom of the chin back to the sizing guide. The perpendicular line will indicate the proper collar size.

Check to Ensure a Proper Fit

- The chin must be flush with the front of the chin support on the front panel.
- The flexible tabs on the front and back panels should be bent slightly outward to promote a better fit.
- The collar should be "snug" but not touch the tracheal area.



Extra Tall

Tall

Regular

Short

IMPORTANT: Proper training and education in the use of this device should take place before it is applied. These instructions and guidelines are not offered as medical recommendations.

69 REV A